Lansinoh.

COLD & WARM

Microwave Instructions			
<div></div> <table> <tbody><tr> <td>25s / 700W</td> <td>19s / 1,000W</td> <td>16s / 1,250W</td></tr> </tbody></table>	25s / 700W	19s / 1,000W	16s / 1,250W
25s / 700W	19s / 1,000W	16s / 1,250W	

EN

Use cold for relief from pain and discomfort after childbirth, c-section or from haemorrhoids. Use warm at the abdomen for soothing relief from postnatal uterine cramping.

For Cold Therapy:

Freeze pack for 4 hours before use.

For Heat Therapy:

Microwave for time indicated at the first page. Gently massage pad to evenly distribute heat and test pack prior to use. Add time in 5 second increments until desired temperature is reached.

For Heat Therapy -

Water Bath Instructions:

Fill a large pot with enough water to fully submerge the pack without touching the bottom of the pot. Bring water to a boil and remove pot from heating surface. **Do not immerse pack while water is being heated.** Use a blunt, nonmetallic tool to submerge pack into the water for 1-2 minutes. **Do not leave pack unattended while heating.**

If pack has not reached desired temperature, submerge again (in same water) for an additional minute. Use blunt, non-metallic tool to remove from water.

Do not allow the pack to come in contact with the pot or heat source for prolonged time, excessive contact may rupture the pack. Towel dry pack before applying. Always inspect the pack for signs of damage or leaking prior to use and discard if damage is found. Always test the pack temperature before application and allow to cool if needed.

Apply pack for 20 minutes or as recommended by your doctor.

Sleeve Instructions

After cooling or heating, place the pack into the disposable sleeve. Sleeves may be used with cold or warm pack. Refill pack of disposable sleeves available separately.

Cleaning Instructions:

After each use, thoroughly clean pack with mild soap and water. Rinse thoroughly until all soap and residue is removed. Allow to air dry completely prior to use. We do not recommend the use of ethanol, phenol, or chlorine-based products

Suggested Uses

Perineal pain and haemorrhoids

Cold therapy, applied to the affected area can help reduce swelling and relieve pain. Apply the pack for up 20 minutes at a time allowing at least an hour in between uses.

C-Section relief

Cold therapy applied to the affected area can help reduce swelling and ease discomfort. Use for up to 20 minutes at a time or as directed by your physician.

Postnatal uterine cramping

Heat therapy, applied for twenty minutes at a time with at least an hour in between or as directed by your physician may help relieve the discomfort of the cramping, as your uterus begins to return to its original size.

CAUTION: Always test pack temperature prior to application. Prolonged application may cause injury, including burns. Do not use on infants. Do not use if you have poor circulation. Do not use heat therapy on stitches or an open wound. Do not fall asleep using pack. Keep out of reach of children. For extreme, prolonged, or chronic pain, contact a physician. Use only as directed. Heating beyond suggested time may damage or destroy pack. Do not puncture pack and/or ingest contents. For external use only. Do not place disposable sleeves in microwave.

		Everbiz GmbH, Landsberger Str. 155 80687, München, Germany
		NPZ Technology Ltd, 8 Saberton Close, Waterbeach, Cambridge, CB25 9QW, United Kingdom

	Serious incidents must be reported to Health Plus (Shanghai) International Co., Ltd. and the local competent authority who will provide further advice.
----------------------------	---

	Lansinoh® is a registered trademark of Lansinoh Laboratories Inc. © 2023 Lansinoh Laboratories Inc.
----------------------------	---

	Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK
----------------------------	---

	Health Plus (Shanghai) International Co., Ltd. Room 1-X3, Building 6, 4299 Jindu Road, Minhang District, Shanghai, China
----------------------------	--

	Do not use if package is damaged. Consult instructions for use.	Made in China PE69130-69140IF0423
----------------------------	---	-----------------------------------

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