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NEW Pump directly into our
breastmilk storage bags from
any Lansinoh® Breast Pump*
*visit: www.lansinoh.co.uk/products/breastmilk-storage-bags-1

www.lansinoh.co.uk

GB Breastmilk – best for your baby

The milk you express for your baby is a precious fluid. It combines the best possible nutrition with antibodies, live cells, and other agents that protect babies from infection and help them grow and develop. You'll want to take good care of it and how you store it affects how well it is preserved.

Lansinoh® Breastmilk Storage Bags are made from food safe polyethylene (PE). They are also BPA and BPS free.

Please handle Lansinoh® Breastmilk Storage Bags with care when storing in and removing from the freezer, and especially when defrosting. If a bag is dropped, or damaged in any way, it may leak while defrosting.

Instructions for use

- Tear off the safety seal at the top of the bag and open by gently pulling the zipper apart.
- Lansinoh® Breastmilk Storage Bags are pre-sterilised and do not need to be washed before use.
- Pour breastmilk into the storage bag.
- Before closing the bag, read the fill quantity by means of the imprinted scale, close the bag and mark quantity in the inscription area.
- Seal the bag by compressing the double seal. Repeat the procedure to ensure safe sealing.

You can now use your Lansinoh® Pump to pump directly into Lansinoh® Breastmilk Storage Bags.

IMPORTANT! Ensure that your Lansinoh® Breastmilk Storage Bag has white and purple flowers and a white line above the logo as shown in the image. If your bag does not have this design, it is not compatible with pumping directly into the bag.

- To do this, tilt expression set and insert one side to align with the printed white line and flower.
- Gently pull the other side of the bag up over the expression set so the clear zipper is above the expression set on all sides.
- Gently pull the bag down until the expression set is just below the clear bottom zipper all the way around the bag. Tug gently on bottom of bag to ensure bag is secure.
- If working with a double pump, repeat steps 1-3.
- Pump as usual

6) To remove bag, reverse step 2. DO NOT PULL BAG STRAIGHT DOWN. Securely hold the bag and gently pull the zipper seal down past expression set. Maintaining a firm grip on bag, tilt and remove adapter. Seal bag.

NOTE: Do not fill past the 6oz / 180ml mark. Overfilling may cause the bag to split due to the liquid expanding when frozen.

Pumping directly into bags requires two hands to safely remove milk storage bags

Storage of breastmilk

Check with your midwife, health visitor or breastfeeding counsellor for specific storage instructions. When you are freezing your breastmilk, do not fill bags more than ¾ full to allow for expansion from freezing. Date bags with the date and time you expressed your milk for future reference, and note volume of breastmilk.

How long can breastmilk be stored?	
Room temperature	6 to 8 hours
Fridge (up to 4° C)	Up to 3 days
Freezer 0° F (-18° C)	6 months

How long can thawed breastmilk be stored?	
Fridge (up to 4° C)	unopened 24 hours
Fridge (up to 4° C)	after opening 12 hours

Source: La Leche League International, 2008

Instructions for defrosting

To preserve the precious substances contained in your breastmilk, defrosting should be carried out as gently as possible – either very slowly over 24 hours in the fridge, or at room temperature. Do not use boiling water or a microwave! If necessary, you can defrost your breastmilk under running cold or lukewarm tap water or in a bottle heater (max 37° C). If the milk settles into different layers, you can mix it by carefully shaking the bag.

After defrosting, you can put the milk back into the fridge until consumption. Unopened, the bag containing defrosted breastmilk can be kept in the fridge for 24 hours.

After opening, defrosted milk must be kept in the fridge and consumed within 12 hours. After warming, the remainder must be discarded.

Notes

DO NOT refreeze thawed breastmilk.

DO NOT store breastmilk in freezer door. The door tends to

be the warmest part of the freezer.

Date all breastmilk and use the oldest breastmilk first.

Single use only.

Bags can be stored upright or flat in the fridge or freezer. When stored flat, bags are easier to thaw.

To avoid freezer burn, as much air as possible should be removed from the storage bags before freezing. To do this, take the bag between your forefinger and middle finger (like scissors) and press the air above the liquid level towards the zipper.

When storing flat, separate each bag with a paper towel to avoid bags sticking together.

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Lansinoh® and HPA®, are registered trademarks of Lansinoh Laboratories, Inc. ©2017 Lansinoh Laboratories, Inc. All rights reserved. Manufactured in Thailand.