


Simple, soothing relief
straight from your freezer

Infused with a moisturising
blend of organic aloe vera
& witch hazel

No added fragrance &
suitable for sensitive skin

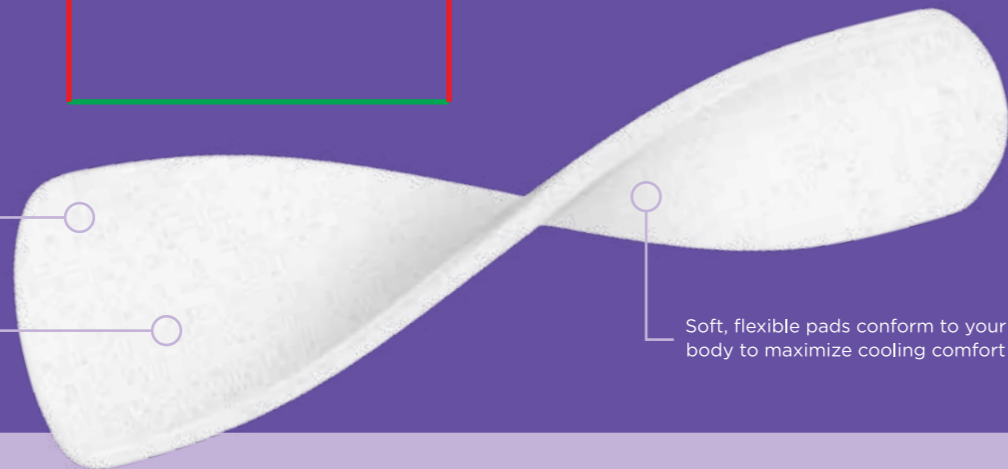

Soft, flexible pads conform to your
body to maximize cooling comfort

Ingredients: Aqua, Glycerin, Hamamelis Virginiana (Witch Hazel) Water, Xanthan gum, Aloe Barbadensis Leaf Juice*, Lactic Acid, Sodium Benzoate, Potassium Sorbate, Benzyl Alcohol, Didecylidimonium Chloride (*Organic Ingredient)

Individually wrapped for hygiene

Instructions

Preparation: Ensure gel is evenly distributed by gently massaging the full length of the pouch several times. Depending on preference, pads may be frozen or refrigerated. Place the unopened pouch flat in the freezer or refrigerator for at least 24 hours.

Application: Wash hands thoroughly. Gently flex the pouch to release the pad before opening. Tear the pouch and carefully remove frozen pad. Test temperature prior to use. Place the pad on top of your maternity pad in direct contact with the skin. Change the pad as frequently as needed. Discard after use.

Precautions: Keep out of reach of children. Do not use on infants or children under 12. For external use only. Do not use on breasts, nipples, in or near mouth. Do not apply to open wounds. If irritation occurs, discontinue use and consult your healthcare professional. Single use product. Store flat in a cool, dark place away from direct sunlight prior to use. Do not heat. For maximum effectiveness, use within 3 months of freezing. Do not refreeze.

Lansinoh.co.uk

Lansinoh.

Postpartum Padsicles

Herbal infused & ready-to-freeze for soothing, cooling relief

Gynecologically & dermatologically tested | 8 pads infused with 22.5g moisturising gel

Lansinoh.

Postpartum Padsicles

Herbal infused & ready-to-freeze for soothing, cooling relief

8 pads infused with 22.5g moisturising gel



Gynecologically &
dermatologically tested

Lansinoh Laboratories UK Ltd
Ground Floor, West Mayesbrook House, Lawnswood Business Park
Leeds, LS16 6QY, UK | Lansinoh.co.uk

Lansinoh Laboratories Inc,
Niederlassung Deutschland, Bayerischer Platz 1
10779 Berlin, Germany

Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc.
© 2024 Lansinoh Laboratories, Inc. All rights reserved.
Made in Türkiye



Lansinoh Laboratorres
Sağlık Gereçleri Tasarım San. Tic. Ltd. Şti.
Zafer SB Mah. Gunduz Sk. No:13 E.S.B
Gazimihir /Izmir/ Türkiye